



Please read the instructions before use.


Kiwi Shield are not suppliers of official medical-grade PPE. This initiative has no official backing by the Ministry of Health or any other government department. So please understand, **these are to be used at your own risk.**


Kiwi shield is a laser-cut PET plastic protection visor as an additional layer to other PPE not a replacement . Due to the laser cut production of these shields, the components may appear dirty on the outer layer of protective film this should not permeate onto the shield once film is removed. **If this does please discard the shield.**

This is an initiative that has been created due to demand of doctors and other health professionals made the issue known in our community.


We recommend the following steps are taken:

- Disinfect the shield upon receipt
- Remove the protective film from both sides of the lasercut cut PET. **This is important as not doing so may damage the shield during assembly.**
- Repeat disinfecting process after assembly.

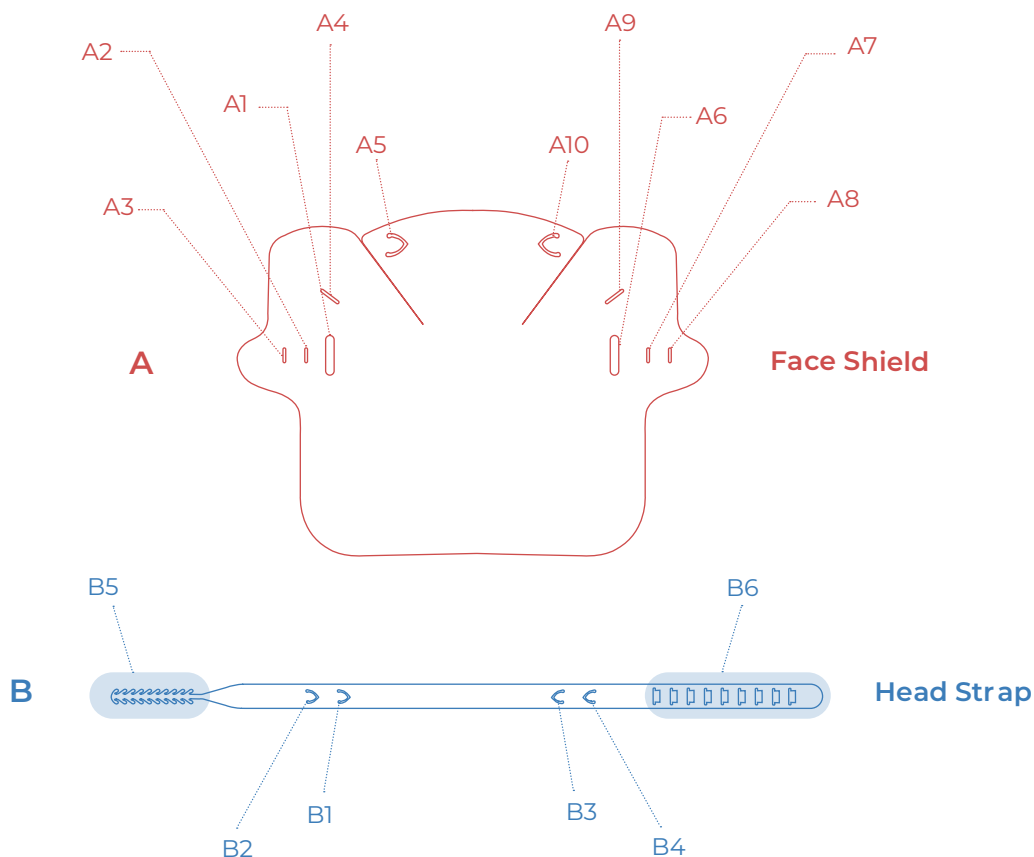


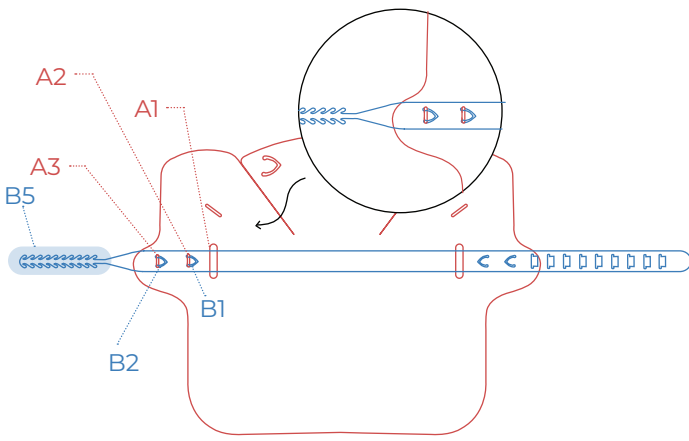


1 Thread **B5** through **A1**, clip **B2** into **A3** and **B1** into **A2**. Pull to seat both tags securely.

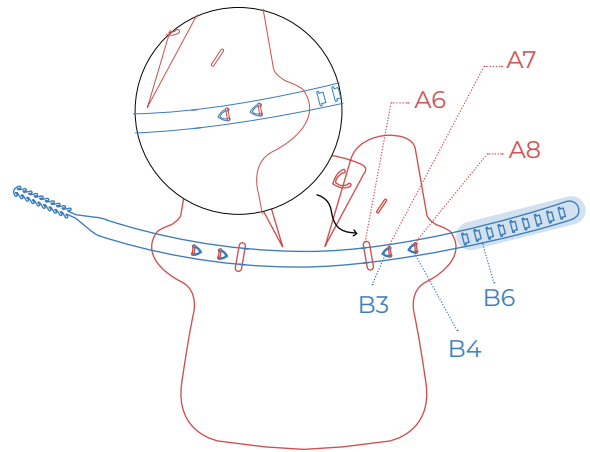


2 Thread **B6** through **A6**, clip **B3** into **A7** and **B4** into **A8**. Pull to seat both tags securely.

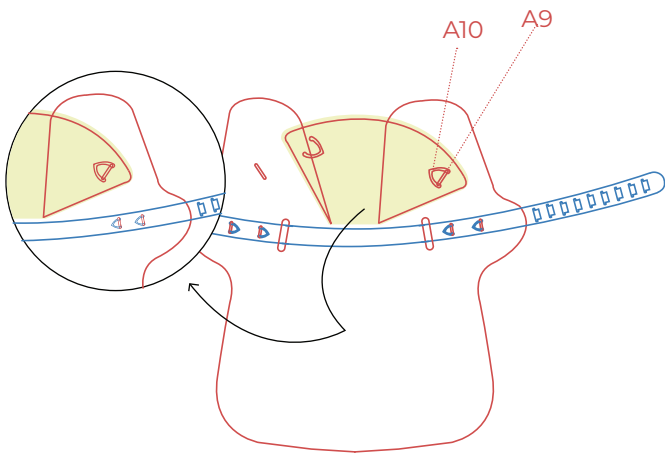


3 Bend **A10** to clip into **A9**. Ensure the inner flap (marked in yellow) sits on top of the exterior flaps.

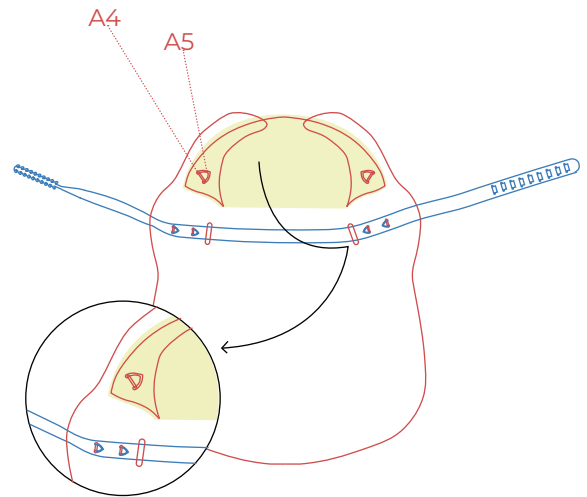

4 Bend **A5** to clip into **A4**. Ensure the inner flap (marked in yellow) sits on top of the exterior flaps.

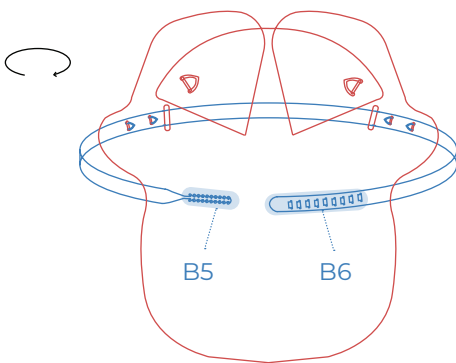

5 Turn the shield around so **B5** and **B6** are facing you.

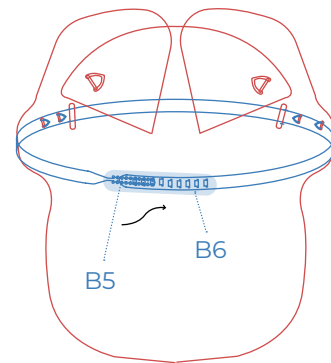

6 Thread **B5** through a slit of **B6**. The size of the head strap will depend on the slit you choose. To adjust pull the barbs tighter or select a smaller slit.